HYDRAFACIAL ANTI-AGING BOOST PLUS

M/HYDRAFACIAL REGEN GF BOOSTER

ReGen GFTM

Skin Perfecting Protein Solution with M-007TM Technology

Benefits

- Regenerative Peptides mimic growth factors to boost production of collagen and elastin.
- Breakthrough M-007TM delivery technology enhances treatments by allowing deeper, more effective absorption.

Ingredient Highlights

- Heptapeptide-32 maintains the appearance of healthy, youthful skin through collagen and elastin production.
- Copper peptide Cu-GHK regenerates healthy-looking skin by accelerating wound healing and skin repair.
- Palmitoyl tetrapeptide-7 treats inflammation while boosting the growth of collagen-producing tissues.
- Palmitoyl tripeptide-5 helps skin cells flush toxins while enhancing the appearance of firmness.
- Azelaoyl bis-dipeptide 10 addresses signs of aging by managing the formation of free radicals.

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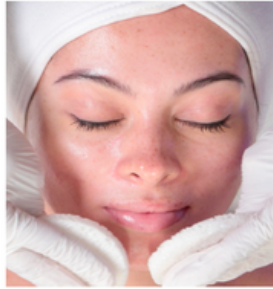
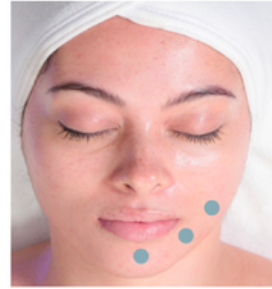
GLYSAL™

BENEFITS

- Combines the benefits of aggressive exfoliation and chemical peels into a one-step resurfacing procedure.
- The result combines accelerated peel benefits with visible skin rejuvenation and no irritation.

INGREDIENT HIGHLIGHTS

- Glycolic Acid – Exfoliator
- Salicylic Acid – Exfoliator

GLYSAL™ PREP

7.5% Glycolic Acid
2% Salicylic Acid Peel
pH Level 3.0–3.5

GLYSAL™ PEEL

15% Glycolic Acid
1.5% Salicylic Acid Peel
pH Level 3.0–3.3

GLYSAL™ PEEL MD

30% Glycolic Acid
1.5% Salicylic Acid Peel
pH Level 3.0–3.3
(Physicians Only)

HYDRAFACIAL ANTI-AGING BOOST PLUS

M/HYDRAFACIAL REGEN GF BOOSTER

TRINN 3: VAKUUM UTSUG - VAKUUM 20-30

Beta-HD Clear

Liten lysegrønn tip

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Sakte fjærlette strøk
overlapp 40-60%

Sakte fjærlette strøk
overlapp 40-60%

Tapping/hold

Sakte fjærlette strøk
overlapp 40-60%

Sakte fjærlette strøk
overlapp 40-60%

Sakte fjærlette strøk
overlapp 40-60%

Sakte fjærlette strøk
overlapp 40-60%

Sakte fjærlette strøk
overlapp 40-60%

Sakte fjærlette strøk
overlapp 40-60%

Sakte fjærlette strøk
overlapp 40-60%

HYDRAFACIAL ANTI-AGING BOOST PLUSS
M/HYDRAFACIAL REGEN GF BOOSTER

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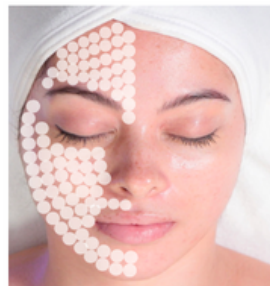
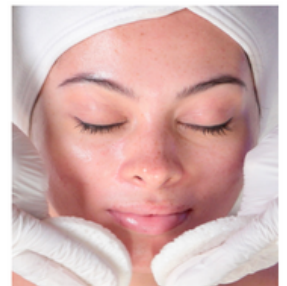
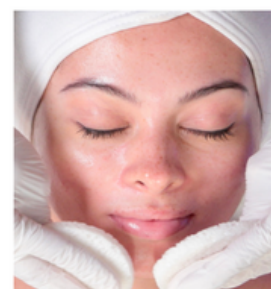
HYDRAFACIAL ANTI-AGING BOOST PLUS
M/HYDRAFACIAL REGEN GF BOOSTER